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FRONT

UMONIUM³⁸ NEUTRALIS

INDICATIONS: Cleaning and disinfection for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. **INSTRUCTIONS:** *Cleaning/Pre-disinfection:* Use at 0.5% (25ml/5L of water*). Soak then scrub/use the ultrasounds or wet wipe to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. *Disinfection:* Use at 2.5% (125 ml/L of water*) on clean devices. Apply by wet wiping or soaking, making sure to respect a minimum contact time of 10 minutes. If you are cleaning invasive medical devices, Rinse with water*. If you are cleaning non-invasive medical device surfaces, leave to dry or rinse with water*. Do not use a cloth dampened with water*. If necessary, *Clean drinking water at room temperature. **PRECAUTIONS:** Pure product may cause irritation to eyes and skin. Avoid skin contact. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Do not discharge pure product into the environment. Consult specific directions and material safety data sheet.

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COMPOSITION: UMONIUM³⁸ NEUTRALIS (5L): N-benzyl-N-dodecyl-N,N-dimethyl-ammonium chloride / N-benzyl-N,N-dimethyl-N-tetradecyl-ammonium chloride, 99g/L.

INDONESIA AUTHORIZED REPRESENTATIVE: PT. ISOTEKINDO INTERTAMA
Jl. Raya Kaligayam Lama No. 309 C, Jakarta 12220, Indonesia
MEDICAL DEVICE CLASS B - No. Reg. Alat Kesehatan KEMENKES RI AEL 000000000000

**laboratoire
HUCKERT'S** INTERNATIONAL
20 Avenue Lavoisier - 1300 Waere - BELGIUM
Tel: +32 (0)67 89 41 00 - www.huckerts.net

BACK

FRONT

BACK

INDICATIONS: Cleaning and disinfection for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. **INSTRUCTIONS:** *Cleaning/Pre-disinfection:* Use at 0.5% (25ml/5L of water*). Soak then scrub/use the ultrasounds or wet wipe to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. *Disinfection:* Use at 2.5% (125 ml/L of water*) on clean devices. Apply by wet wiping or soaking, making sure to respect a minimum contact time of 10 minutes. If you are cleaning invasive medical devices, Rinse with water*. If you are cleaning non-invasive medical device surfaces, Leave to dry or rinse with water* (or using a cloth dampened with water*) if necessary. *Clean drinking water at room temperature. **PRECAUTIONS:** Pure product may cause irritation to eyes and skin. Avoid skin contact. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Do not discharge pure product into the environment. Consult specific directions and material safety data sheet.

COMPOSITION: UMONIUM³⁸ NEUTRALIS (5L): N-benzyl-N-dodecyl-N,N-dimethyl-ammonium chloride / N-benzyl-N,N-dimethyl-N-tetradecyl-ammonium chloride, 99g/L.

125 ml e 4.4 oz.fl. 25ml = 2X 12,5ml

INDONESIA AUTHORIZED REPRESENTATIVE: PT. ISOTEKINDO INTERTAMA
Jl. Raya Kaligayam Lama No. 309 C, Jakarta 12220, Indonesia
No. Reg. Alat Kesehatan KEMENKES RI AEL 000000000000

**laboratoire
HUCKERT'S** INTERNATIONAL
20 Avenue Lavoisier - 1300 Waere - BELGIUM
Tel: +32 (0)67 89 41 00 - www.huckerts.net

ETW401 - Ed. 10 - 05/2024

ETW401 - Ed. 10 - 05/2024