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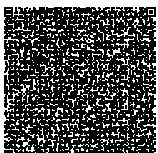
Sinocare

DB / TP / ALP / GGT

Direct Bilirubin/Total Protein/Alkaline Phosphatase/  
Glutamyl Transferase Reagent Kit

12

Direct Bilirubin/Total Protein/Alkaline Phosphatase/  
Glutamyl Transferase Reagent Kit

DB / TP / ALP / GGT

12

Specifications

[ Package Size ]  Specification A: 30 tests/kit  Specification B: 30 tests/kit  
[ Storage Conditions & Expiry date ]

Product	Expired by	Storage Conditions
Reagent cartridge	2 hours	Room temperature. Before testing, please equilibrate the reagent cartridge to room temperature. (Avoid bright light exposure) Cartridges cannot be returned to the refrigerator once they had been at room temperature.
	12 months	2-8°C.

If applicable

Product	Expired by	Storage Conditions
Control Material	DIBALP	7 days
	TPICGT	14 days
	DB	14 days
	TPICGTALP	28 days
Unopened		12 months
		2-8°C.

[ Test environment ]  
Temperature: 15°C -30°C  
Relative Humidity: 25%-85%  
Please perform the test under prescribed conditions, or incorrect results may be obtained.

KEMENKES RI AKL  
Didistribusikan oleh:  
PT Sinocare Healthcare Indonesia  
Jakarta - Indonesia

Direct Bilirubin/Total Protein/Alkaline Phosphatase/  
Glutamyl Transferase Reagent Kit

DB / TP / ALP / GGT

12

Sinocare

The Direct Bilirubin/Total Protein/Alkaline Phosphatase (Glutamyl) Transferase Reagent Kit is intended to quantitatively determine the concentration of direct bilirubin (DB) and total protein (TP), and the activity of alkaline phosphatase (ALP) and γ-glutamyl transferase (GGT) in human serum. Clinically, it is mainly used for auxiliary diagnosis of liver diseases. [Applicable Device] Portable Automatic Multi-function Analyzer

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
Cautions

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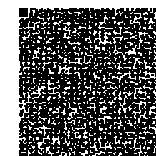
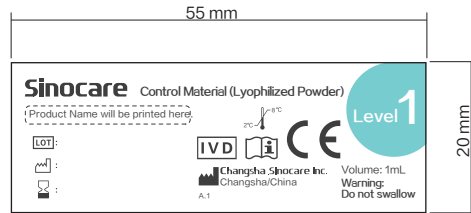
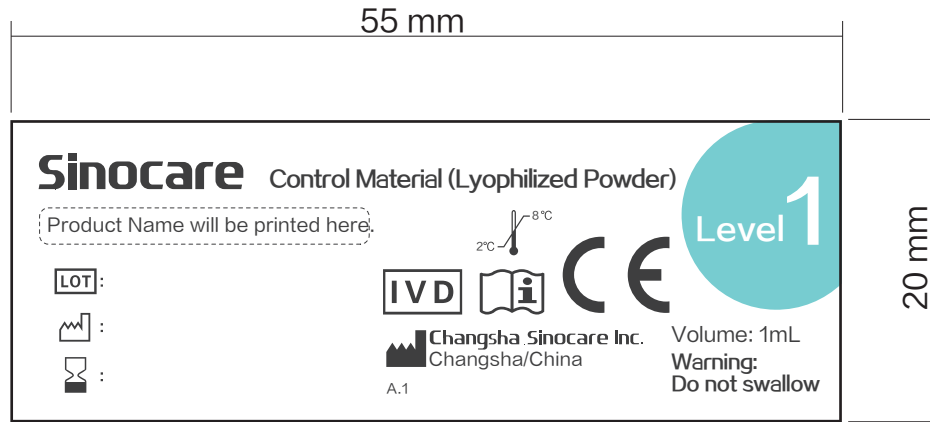
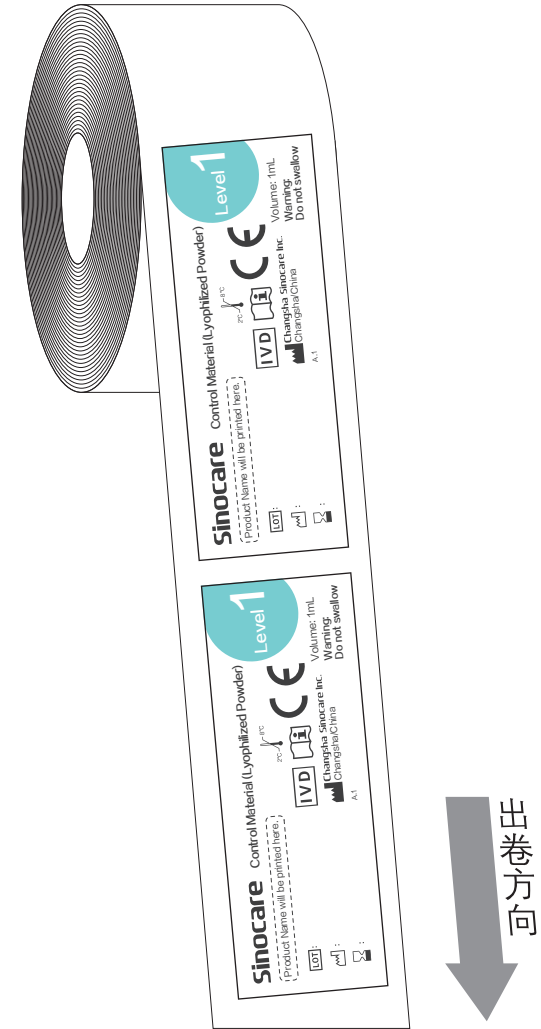
- This product is for in vitro diagnostic use only. Please take protective measures during operation (ex: wearing mask, gloves, etc.). Once contact with eyes or skin, wash immediately with water; go to the hospital if swallow by mistake;
- Please treat used or expired reagent kits as medical waste when disposing them.

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