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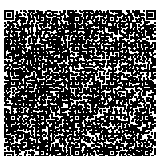
Ir. Sodikin Sadek, M.Kes
NIP. 19621203 198603 1 004


GAMBAR KEMASAN PRIMER


	<p>LOT Lot No.:</p> <p>Vial Open Date:</p> <p>Discard Date:</p>	<p>mALB/Cr Control Solution For Sinocare ACR Monitoring System</p> <p>WARNING: Do not ingest.</p> <p>3ml</p> <p>Please refer to the label on the packing box for date of manufacture & expiry date.</p> <p>Once opened, please use within 1 month. For detailed information, please refer to package insert.</p>	<p>Level 1</p>
<p>Changsha Sinocare Inc. No. 265, Guyuan Road, Hi-Tech Zone, Changsha, Hunan, China QO-PQ-VPQ</p>			

	<p>LOT Lot No.:</p> <p>Vial Open Date:</p> <p>Discard Date:</p>	<p>mALB/Cr Control Solution For Sinocare ACR Monitoring System</p> <p>WARNING: Do not ingest.</p> <p>3ml</p> <p>Please refer to the label on the packing box for date of manufacture & expiry date.</p> <p>Once opened, please use within 1 month. For detailed information, please refer to package insert.</p>	<p>Level 2</p>
<p>Changsha Sinocare Inc. No. 265, Guyuan Road, Hi-Tech Zone, Changsha, Hunan, China QO-PQ-VPQ</p>			

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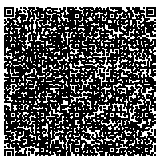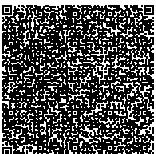Product	Expired by		Storage Conditions
Reagent Kit	Opened	2 hours	Keep away from sunlight; Before testing, please balance the temperature of reagent to room temperature.
	Unopened	12 months	
Control Solution	Opened	1 month	Store at 2-8°C; Do not freeze.
	Unopened	12 months	

【 Test Environment 】 15°C ~ 35°C, 15% ~ 75% RH
Please perform the test under prescribed conditions, or incorrect results may be obtained.

【 Cautions 】

1. For in vitro diagnostic use only.
2. Do not use expired reagent kit.
3. The liquids in the reagent kit are chemical reagents, please do not drink; Avoid contacting with eyes and skin, once contact with eyes or skin, wash immediately with water, ask for the help from doctor if necessary.
4. Follow the national/local rules and regulations to dispose the used reagent kits and dropper pipettes.
5. The reagent kits and disposable dropper pipettes are not reusable.
6. The mALB/Cr Reagent Kit is only applicable for the supporting devices, e.g: PABA-1000 ACR Analyzer.

Penjelasan lay out kemasan dus sampling

GAMBAR PRODUK