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EN INDICATIONS: Cleaning and disinfection for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. Transport of soiled instruments. **INSTRUCTIONS:** **Transport:** Set the spray to the "foam" position and cover the devices with the foam. Make sure to cover it well to limit evaporation. **Cleaning/Pre-disinfection:** Set the spray to the "foam" position. Spray and scrub the devices to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. **Disinfection:** Set the spray to the "fine spray" position. Spray over clean devices. Ensure a minimum contact time of 5 minutes. If you are cleaning invasive medical devices: Rinse with water*. If you are cleaning non-invasive medical device surfaces: Leave to dry or rinse with water* (or using a cloth dampened with water*) if necessary. *Clean drinking water at room temperature. **PRECAUTIONS:** Handle in accordance with good industrial hygiene and safety procedures. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Consult specific directions and material safety data sheet.

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250 mL e 8,8 oz.fl. **COMPOSITION** UMONIUM³⁸ **MEDICAL SPRAY** (AL) : N-benzyl-N-dodecyl-N,N-dimethyl-ammonium chloride/N-benzyl-N, N-dimethyl-N-tetradecyl-ammonium chloride. 3,3 g/L

INDONESIA AUTHORIZED REPRESENTATIVE: PT. ISOTEKINDO INTERTAMA

Jl. Raya Kebayoran Lama No. 309 C, Jakarta 12210, Indonesia

MEDICAL DEVICE CLASS B – No. Reg. Alat Kesehatan KEMENKES RI AKL XXXXXXXXXX

LABORATOIRE HUCKERT'S INTERNATIONAL
20 Avenue Lavoisier - 1300 Wavre - BELGIUM
Tel: +32 (0)67 89 41 00 - www.huckerts.net

500 mL

UMONIUM³⁸ MEDICAL SPRAY

DISINFECTANT, CLEANER SPRAY

FOR MEDICAL DEVICES

DISINFECTS FROM 30 SECONDS

DISINFECTAN, SEMPROTAN PEMBERSIH

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MADE IN BELGIUM

BACTERICIDAL, MYCOBACTERICIDAL, FUNGICIDAL, VIRUCIDAL
EN11040:2006, EN1275:2005, EN13624:2004, EN13727:2003, EN14348:2005,
EN14476:2013, EN14561:2006, EN14562:2006, EN14563:2009

EN INDICATIONS: Cleaning and disinfection for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. Transport of soiled instruments. **INSTRUCTIONS:**

Transport: Set the spray to the "foam" position and cover the devices with the foam. Make sure to cover it well to limit evaporation. **Cleaning/Pre-disinfection:** Set the spray to the "foam" position. Spray and scrub the devices to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. **Disinfection:** Set the spray to the "fine spray" position. Spray over clean devices. Ensure a minimum contact time of 5 minutes. If you are cleaning invasive medical devices: Rinse with water*. If you are cleaning non-invasive medical device surfaces: Leave to dry or rinse with water* (or using a cloth dampened with water*) if necessary. **Clean drinking water at room temperature. PRECAUTIONS:**

Handle in accordance with good industrial hygiene and safety procedures. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Consult specific directions and material safety data sheet.

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500 ml e 17,6 fl.oz

ET00262 - Ed 1.0 - 10/2021

COMPOSITION UMONIUM³⁸ MEDICAL SPRAY : (AL)

N-benzyl-N-dodecyl-N,N-dimethyl-ammonium chloride/N-benzyl-N,

N-dimethyl-N-tetradecyl-ammonium chloride. 3.3 g/L

INDONESIA AUTHORIZED REPRESENTATIVE: PT. ISOTEKINDO INTERTAMA.
Jl. Raya Kebayoran Lama No. 309 C, Jakarta 12210, Indonesia
MEDICAL DEVICE CLASS B - No. Reg. Alat Kesehatan KEMENKES RI AKL XXXXXXXXXX

LABORATOIRE HUCKERTS INTERNATIONAL
20 Avenue Lavoisier - 1300 Wavre - BELGIUM
Tel.: +32 (0)67 89 41 00 - www.huckerts.net

1L

UMONIUM³⁸ MEDICAL SPRAY

DISINFECTANT, CLEANER SPRAY FOR MEDICAL DEVICES
DISINFECTS FROM 30 SECONDS

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EN INDICATIONS: Cleaning and disinfection for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. **Transport of soiled instruments.**
INSTRUCTIONS: Transport: Set the spray to the "foam" position and cover the devices with the foam. Make sure to cover it well to limit evaporation. **Cleaning/Pre-disinfection:** Set the spray to the "foam" position. Spray and scrub the devices to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. **Disinfection:** Set the spray to the "fine spray" position. Spray over clean devices. Ensure a minimum contact time of 5 minutes. If you are cleaning invasive medical devices: Rinse with water*, if you are cleaning non-invasive medical device surfaces: Leave to dry or rinse with water* (or using a cloth dampened with water*) if necessary. ***Clean drinking water at room temperature.**
PRECAUTIONS: Handle in accordance with good industrial hygiene and safety procedures. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Consult specific directions and material safety data sheet.

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11E 35,2 oz.fl.

ET00261 - Ed 1.0 - 10/2021

COMPOSITION UMONIUM³⁸ MEDICAL SPRAY: (AL)
N-benzyl-N-dodecyl-N,N-dimethyl-ammonium chloride/N-benzyl-N,N-dimethyl-N-tetradecyl-ammonium chloride. 3.3 g/L.

INDONESIA AUTHORIZED REPRESENTATIVE: PT. ISOTEKINDO INTERTAMA.
Jl. Raya Kebayoran Lama No. 309 C, Jakarta 12210, Indonesia
MEDICAL DEVICE CLASS B - No. Reg. Alat Kesehatan KEMENKES RI AKL XXXXXXXXXX