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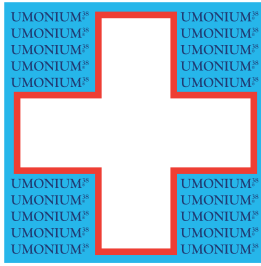
BACTERICIDAL, MYCOBACTERICIDAL, FUNGICIDAL, VIRUCIDAL

EN1040:2006, EN1275:2005, EN13624:2004, EN13727:2003, EN14348:2005, EN14476:2013, EN14561:2006, EN14562:2006, EN14563:2009

MADE IN BELGIUM
1l e 35.2 fl.oz

BACK

EN Cleaning and disinfection for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. **INSTRUCTIONS: Cleaning/Pre-disinfection:** Use at 0.5% (25mL/5L of water*). Soak then scrub/use the ultrasounds or wet wipe to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. **Disinfection:** Use at 2.5% (25 mL/L of water*) on clean devices. Apply by wet wiping or soaking, making sure to respect a minimum contact time of 10 minutes. If you are cleaning invasive medical devices: Rinse with water*. If you are cleaning non-invasive medical device surfaces: Leave to dry or rinse with water* (or using a cloth dampened with water*) if necessary. *Clean drinking water at room temperature. **PRECAUTIONS:** Pure product may cause irritation to eyes and skin. Avoid skin contact. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Do not discharge pure product into the environment. Consult specific directions and material safety data sheet.

ETOV401 - Ed 1.0 - 05/2024

COMPOSITION UMONIUM³⁸ **NEUTRALIS** (SL): N-benzyl-N-dodecyl-N,N-dimethyl-ammonium chloride / N-benzyl-N,N-dimethyl-N-tetradecyl-ammonium chloride. 99g/L.

125 ml e 4,4 oz.fl. **25ml** = 2X **12,5ml**

INDONESIA AUTHORIZED REPRESENTATIVE: PT. ISOTEKINDO INTERTAMA.
Jl. Raya Kebayoran Lama No. 309 C, Jakarta 12210, Indonesia
No. Reg. Alat Kesehatan KEMENKES RI AKL XXXXXXXXXXXX


LABORATOIRE HUCKERT'S INTERNATIONAL
20 Avenue Lavoisier - 1300 Wavre - BELGIUM
Tel: +32 (0)67 89 41 00 - www.huckerts.net

1 L

FRONT

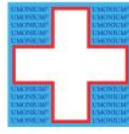ETOR300 - Ed 1.0 - 10/2021

UMONIUM³⁸ NEUTRALIS

DISINFECTANT, CLEANER COLOURLESS AND ODOURLESS
DISINFECTAN, PEMBERSIH TANPA WARNA DAN BAU

BACTERICIDAL, MYCOBACTERICIDAL, FUNGICIDAL, VIRUCIDAL

EN1040:2006, EN1275:2005, EN13624:2004, EN13727:2003, EN14348:2005, EN14476:2013, EN14561:2006, EN14562:2006, EN14563:2009

5 425037 713863

MADE IN BELGIUM
1l e 35.2 fl.oz

BACK

UMONIUM³⁸ NEUTRALIS

EN INDICATIONS: Cleaning and disinfection for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. **INSTRUCTIONS: Cleaning/Pre-disinfection:** Use at 0.5% (25mL/5L of water*). Soak then scrub/use the ultrasounds or wet wipe to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. **Disinfection:** Use at 2.5% (25 mL/L of water*) on clean devices. Apply by wet wiping or soaking, making sure to respect a minimum contact time of 10 minutes. If you are cleaning invasive medical devices: Rinse with water*. If you are cleaning non-invasive medical device surfaces: Leave to dry or rinse with water* (or using a cloth dampened with water*) if necessary. *Clean drinking water at room temperature. **PRECAUTIONS:** Pure product may cause irritation to eyes and skin. Avoid skin contact. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Do not discharge pure product into the environment. Consult specific directions and material safety data sheet.

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COMPOSITION UMONIUM³⁸ NEUTRALIS (SL): N-benzyl-N-dodecyl-N, N-dimethyl-ammonium chloride/N-benzyl-N, N-dimethyl-N-tetradecyl-ammonium chloride. 99g/L.

INDONESIA AUTHORIZED REPRESENTATIVE: PT. ISOTEKINDO INTERTAMA.
Jl. Raya Kebayoran Lama No. 309 C, Jakarta 12210, Indonesia
MEDICAL DEVICE CLASS B - No. Reg. Alat Kesehatan KEMENKES RI AKL XXXXXXXXXXXX

LABORATOIRE HUCKERT'S INTERNATIONAL
20 Avenue Lavoisier - 1300 Wavre - BELGIUM
Tel.: +32 (0)67 89 41 00 - www.huckerts.net

ETOV294 - Ed 1.0 - 10/2021

5 L

FRONT

ET0R301 - Ed. 1.0 - 10/2021

UMONIUM³⁸ NEUTRALIS

DISINFECTANT, CLEANER COLOURLESS AND ODOURLESS

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